

Size Compatibility of Airway Equipment in Children

LMA Unique ETT Maximum Size

Size 1	3.0 ID
Size 1 ½	3.5 ID
Size 2	4.5 ID
Size 2 ½	5.0 ID
Size 3	5.5 ID

- *Maximum sizes of endotracheal tubes listed apply to both Portex uncuffed and Kimberly-Clark Microcuff endotracheal tubes*
- *Pilot balloon of cuffed tubes does not pass through LMA less than size 3*

Mallinckrodt Satin-Slip Intubating Stylets

Small	2.0 ETT and larger
Medium	3.5 ETT and larger
Large	5.0 ETT and larger

Frova Intubating Introducer (bougie)

8 FR	3.0 ETT and larger
14 FR	5.0 ETT and larger

Cook Airway Exchange Catheters

8 FR	3.0 ETT and larger
11 FR	4.0 ETT and larger
14 FR	5.0 ETT and larger

- *Internal diameter of 8Fr and 11Fr too small for Storz 2.85mm fibrescope*

Fibreoptic scopes

Storz 2.85 mm scope	3.5 ETT and larger
Storz 3.7 mm scope	4.5 ETT and larger
Storz 5.2 mm scope	6.0 ETT and larger
Pentax 3.4 mm scope	4.0 ETT and larger (<i>Pentax fibrescopes are kept in Main Theatre Suite</i>)